

Retainer **BRITE**[®] Cleaning Tablets

Use daily for best results.



Drop one tablet into enough warm (not hot!) water to cover appliance.



Place appliance into bubbling blue solution.



When blue clears (15-20 minutes**) appliance is clean and odor free!



Rinse appliance and hands thoroughly with running water.

Order your supply of Retainer Brite[®] today!

1-800-866-6884 • www.retainerbrite.com

STORAGE: Place in a cool dry place.

WARNINGS: Contains persulfates which are a known allergen. Keep out of reach of children. Use only as directed. Do not put tablets or cleaning solution in mouth. If ingested, seek medical assistance immediately. In the U.S.A. please call Poison Control Center at 1-800-222-1222. Within the UK, please call NHS Direct at 0845 4647.

CAUTION: Keep out of reach of children. Do not put tablets or cleaning solution in mouth.

INGREDIENTS: Potassium Persulfate Compound, Sodium Perborate, Sodium Bicarbonate, Sodium Sulfate, Sodium Carbonate, Pentasodium Triphosphate, Corn Syrup Solids, Sodium Lauryl Sulfoacetate, PEG-180, Flavor, Magnesium Stearate, Tetrasodium EDTA, FD&C Blue #2 (CI 73015), FD&C Blue #2 Aluminum Lake (CI 73015)

* Independent 2009 U.S. laboratory testing (ASTM method E1153)

**DO NOT allow wire appliances to soak for more than 15 minutes. Remove promptly and rinse thoroughly with water.

©2011 DENTSPLY • 1912 44th Avenue East • Bradenton, FL 34203
RTE-105-11 Rev. 8/2011 #RB-TP-BAG

